CPAP DREAM / ORTO DREAM /  
SPACER

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***INSTRUCTIONS FOR USE***

***KASUTUSJUHEND***

***UPUTE ZA UPORABU***

***NAUDOJIMO INSTRUKCIJA***

***LIETOŠANAS INSTRUKCIJA***

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***NÁVOD NA POUŽITIE***

***NAVODILA ZA UPORABO***

***BRUKSANVISNING***

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Orthopaedic pillow

Product code: ORTO DREAM

Two-piece orthopaedic pillow for lower limbs

Product code: SPACER

### 1. Material

#### CPAP DREAM

With its profiled design and due to the use of shape memory foam, the pillow ensure comfort for CPAP device users. The special design helps prevent the mask from moving during sleep. The pillow is delivered with a pillowcase with a zipper, made of high-quality soft fabric and featuring good air permeability.

#### ORTO DREAM

The pillow is made of shape memory foam to ensure proper support and correct position of the cervical spine and head during sleep. The pillow is delivered with a soft pillowcase with a zipper, made of high-quality soft fabric and featuring good air permeability.

#### SPACER

The two-piece pillow is made of shape memory foam, so it easily adapts to the body and improves user comfort. It can be used in closed and unfolded form. The pillow is delivered with a soft pillowcase with a zipper, made of high-quality soft fabric and featuring good air permeability.

### 2. Application

#### CPAP DREAM

The pillow is designed for sleep apnea sufferers, using a CPAP device. It enables comfortable positioning of the head along with the mask during sleep apnea treatment therapy. For comfortable fitting of the mask the pillow features special cutouts and the shape memory foam in the pillow supports the head, adjusting to the user.

Using the pillow has a positive effect on problems resulting from the incorrect position of the neck and head during sleep (back pain, muscle tension, tingling and numbness in the limbs, etc.). Recommended as healthy sleep prophylaxis.

#### ORTO DREAM

The pillow enforces physiologic alignment of the cervical spine, properly adjusted to the cervical lordosis eliminates muscle tension in the neck area. Designed for people who require stable support for the head and cervical spine during sleep, while reducing pressure on the body. Using the pillow has a positive effect on problems resulting from the incorrect position of the neck and head during sleep (back pain, muscle tension, tingling and numbness in the limbs, etc.). Recommended as healthy sleep prophylaxis.

## SPACER

The pillow is used in knee joint pain conditions and after hip joint alloplasty. In closed form, the pillow properly positions the pelvis and hips, which reduces tension, pain and fatigue in the lower limbs. When used in its unfolded form, it provides support for the lower part of the legs in order to properly position the user.

The cushion ensured physiological alignment of the joints and improves circulation in the lower limbs, and is also used for the prevention of bedsores.

### 3. Contraindications

Users with serious medical conditions or established lesions of the cervical spine should consult a physician or physiotherapist prior to first use of the pillow (applies to CPAP DREAM and ORTO DREAM pillow)

### 4. Remarks

**Timago International Group is not be responsible for improper maintenance of the product, failure to observe safety rules, or using the product contrary to its intended purpose.**

### 5. Technical parameters

Model	CPAP DREAM	ORTO DREAM	SPACER
Size	50 x 30 x 10 cm	60 x 40 x 12 cm	25 x 20 x 15 cm
Density	45D	45D	45D
Composition parts	Pillowcase: Cotton 90%, Polyester 10% Filling: Thermoelastic foam	Pillowcase: Polyester 100% Filling: Thermoelastic foam	Pillowcase: Polyester 100% Filling: Thermoelastic foam
Pillowcase:	vapour-permeable, zipper	vapour-permeable, zipper	vapour-permeable, zipper
Colour	White	White	Grey

### 6. Maintenance

- Wash the pillowcase by hand in lukewarm water, do not spin, do not chlorinate, do not iron, do not bleach, do not tumble dry, do not dry clean.
- Dry at room temperature away from intense heat sources.

**Please note!** Maintenance information can be found on the fabric label.

### 7. Storage and shipment conditions

Do not expose the product to direct sunlight, water and moisture.

## 8. Guarantee information

The product is covered by a guarantee, the terms of which are described in the guarantee card you received. Please retain the proof of purchase (receipt or invoice) for guarantee purposes.

## 9. Markings

	Reference number		Manufacturer
	Series number		Date of manufacture
	Serial number		Medical device
	Remark		Read the user manual
	The manufacturer has assessed conformity with the essential requirements for medical devices.		

The entire offer is available to you from our distributors. Learn more at [www.timago.com](http://www.timago.com)  
Thank you for choosing Timago!

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